

Solidea Sizes 7 – Tights

Products Covered by this size chart

- Solidea Venere 30, Venere 70 and Venere 140
- Solidea Naomi 140
- Solidea Wonder Model 140
- Solidea Alisea, Babylon and Imbrulia
 - Not available in Extra-Extra-Large (XXL)
- Solidea Marilyn
 - Not available in Extra-Large (XL) or Extra-Extra-Large (XXL)

Correct size choice may be made from the ‘leg circumference’ chart below, or the ‘height and weight’ chart on the second page.

Leg Circumferences

Procedure

Follow the measurement Guidelines below.

Choose the correct size from the Size Chart.

Click Close to return to product details and select the correct size

Measurement Guidelines

In order to work effectively graduated compression hosiery must be the correct size and apply the right amount of pressure to the leg. It is essential that accurate measurements are taken.

- All measurements should be taken on the bare leg, in centimetres, whilst standing (if possible).
- Ensure that the tape measure is snug but not too tight.
- Measurements should be taken when the leg is rested and not swollen


Where to Measure?

cG - Widest point of the Thigh

cD - Just above the Knee

cB - Widest point around the Ankle Bone

All measurement's are in cm's

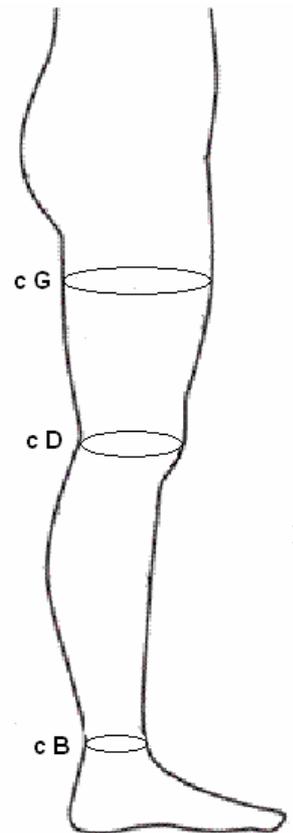

Circumference	1-S	2-M	3-ML	4-L	4-XL	5-XXL
cG	50-57	53-60	56-63	60-67	54-71	68-75
cD	29-34	32-37	35-40	38-43	41-46	44-49
cB	19-21	20-22	22-24	24-26	26-28	28-31

Height and Weight Chart

In some cases, customers might prefer to choose the most appropriate size by referring to the following table; where the various sizes are defined according to a ratio between height in centimeters (HEIGHT CM) and weight kilograms (WEIGHT KG).

If, for example, we consider a person who is 163 cm tall and weighs 60 kilograms, then we calculate as follows:

- We find the corresponding height on the column at the left of the table (HEIGHT CM). In this case 163 cm is a height between 160 and 165 cm
- We look for the corresponding weight in kg on the top of the table (WEIGHT KG). In this case we consider the box that reaches up to 60 kg
- The recommended size is found at the intersection between the column that corresponds to weight and the line that corresponds to height. In our example the size is M (medium).

		WEIGHT KG												
		45	50	55	60	65	70	75	80	85	105			120
HEIGHT CM	145/150	1										4'10"	HEIGHT IN	
	150/155											S		
	155/160	2												
	160/165											M		
	165/170	3												
	170/175											ML		
	175/180	4												
	175/190											L		
		WEIGHT LBS												
		100	110	120	130	140	150	160	170	180	220	250		
		4XL												
												XL		
		5XXL												
												XXL		